Pure Charge&Go X

Guide for Hearing Care Professionals

Made for

iPhone | iPad | iPod

Content

Introduction	3
Before and after fitting	4
Assembling	4
Activating	4
Charging	5
Connecting	5
Noting the serial number	6
Replacing the housing	7
Disassembling	7
Assembling	9
Important safety information	11
General warnings	11
Defective power cells	12
Further information	13
Storage periods	13
Symbols used in this document	13

Introduction

This guide is intended for Hearing Care Professionals. It provides important information on hearing instruments with permanently built-in power cells (lithium-ion batteries).

CAUTION


It is important to read this guide, the user guide and the safety manual thoroughly. Follow the safety information to avoid damage or injury.

Before and after fitting

You have to prepare the hearing instruments before you can start fitting.

Assembling

- ▶ A power cell is installed in every hearing instrument on delivery. There is no need to insert a battery.
- ▶ Attach a suitable ear piece to the suitable receiver unit.
- ▶ Attach the receiver unit to the hearing instrument by clicking it in place.
- ▶ Push the receiver locking bar into the housing.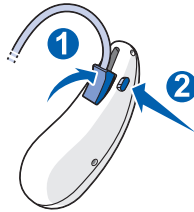

Activating

Before first use or first fitting you have to activate the hearing instruments by putting them into the charger. Wait until the first orange LED lights up.

Note:

- Hearing instruments that are activated and **not used** may discharge faster than hearing instruments that have **not** been activated. Only activate the hearing instruments before first use or first fitting.

Charging

The hearing instruments are pre-charged on delivery. They should have sufficient power for a first fitting session.

We recommend to fully charge the hearing instruments before first fitting and handover to the client. A full charging cycle takes approximately 4 hours.

Follow the operating conditions in both the hearing instrument's and the charger's user guide.

Connecting

Use Noahlink Wireless to fit the hearing instruments wirelessly.


Noting the serial number

Both serial number and performance level are printed on the frame.

When a hearing instrument is returned from our service check the serial number on the frame:

- Remove the rocker switch with your fingernail to see the serial number.

Note: If you press the lower part of the rocker switch more than 3 seconds the hearing instrument may turn on. After reassembly you can turn off the hearing instrument by a long press to the upper part of the rocker switch.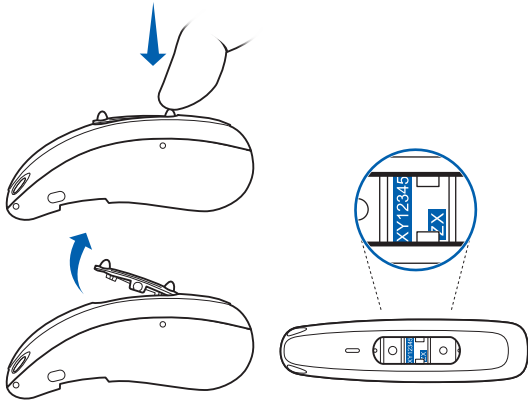

Replacing the housing


NOTICE


- ▶ This maintenance task must be performed by the Hearing Care Professional.

Disassembling


- ▶ Remove the receiver unit and the axles.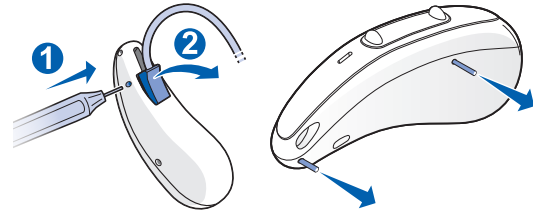

- ▶ Insert the included tool into the front end of the hearing instrument and detach the top housing.

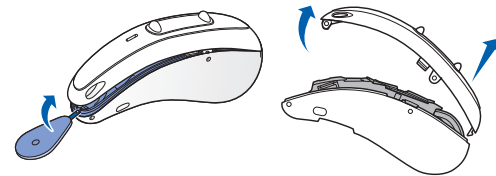
- ▶ Pull off the bottom housing. No tool is required.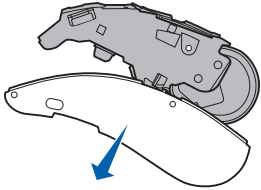


Assembling


- ▶ Install the bottom housing.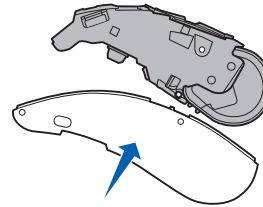

- ▶ Install the top housing from the front.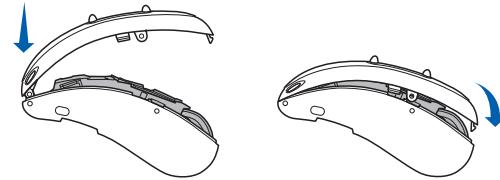

- ▶ Attach the axles. Note the different lengths and diameters of the axles.

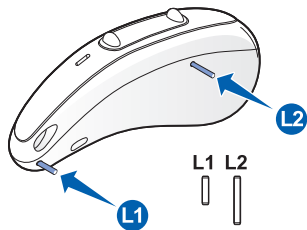- ▶ Attach the receiver unit.

Important safety information

This section contains important safety information regarding the power cell. For more safety information, refer to the safety manual and the user guide supplied with the hearing instruments.

NOTICE

- ▶ Do not exchange power cells. This may only be performed by Service Center only.

General warnings

WARNING

A power cell (lithium-ion rechargeable battery) is permanently built into the hearing instrument. Incorrect use of the power cell or the hearing instrument can cause the power cell to burst. Risk of injury, fire or explosion!

- ▶ Follow the safety instructions for power cells in this section.

Defective power cells

In case you think the power cell is defective comply with the following steps:

- ▶ Place the hearing instrument into the charger. If you get a red LED then the power cell is defective.
- ▶ Perform visual inspection of the hearing instrument. The test is considered failed if it has any of below criteria:
 - ▷ There is physical sign of deformation caused by bloated lithium power cell inside.
 - ▷ There is visible leakage from the lithium power cell.
 - ▷ Product is generating heat on its own or hot to the touch.

If any of the test steps failed, consider the power cell defective. Due to safety concerns the hearing instrument cannot be shipped anymore. Please record the serial number of the hearing instrument, take a photo of the damages, and consult with Sivantos Customer Service team before disposal of the product according to local regulations.

Further information

Storage periods

The shelf life of a hearing instrument which has not been activated is 3 months.

Once a power cell is activated, in order for it to remain in good condition it needs to be charged at least every 6 months. We recommend to charge the hearing instruments every 3 months.

Symbols used in this document

 Points out a situation that could lead to serious, moderate, or minor injuries.

 Indicates possible property damage.

 Advice and tips on how to handle your device better.

Made for
iPhone | iPad | iPod

"Made for iPhone", "Made for iPad", and "Made for iPod" mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

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© 09.2019, Sivantos Inc. All rights reserved.
Sivantos, Inc.