

# Charge&Go Activation for Inductive Charger II



Life sounds brilliant.

Thank you for your purchase of the Charge&Go.

Your new hearing instruments have arrived in sleep mode and will need to be activated for the first time by following these steps:

1. Plug the inductive charger into a power supply.
2. Confirm one green LED for each charging slot which indicates the charger has power and is ready for use.
3. Put the hearing instruments with attached receiver in the charging slots.
4. Wait until you see the "Charge status" change to at least a single orange light.
5. You can now remove your hearing instruments for programming.

## LED Status description

---

 **Orange LED:**  
Charging status 0 to 33 %

---

 **Single green LED:**  
Charging status 34 to 66 %

---

 **Two green LEDs:**  
Charging status 67 to 99 %

---

 **Three green LEDs:**  
Hearing instrument is fully charged.

---

**Please note that this device does not have a wired programming option so it can only be programmed wirelessly (i.e. NoahLink Wireless).**